

# ATHENA CHAMPION™

## *Athena Champion High Speed Turbine (Canister Style)*

*Item Number: 61900*

- *Standard or Fiber Optic*
- *High-quality and Made in the USA*
- *Replacement turbine for Athena Handpiece*
- *Replaces Pana-air type cartridges*

## INSTALLATION INSTRUCTIONS

*Read these instructions before proceeding to replace the turbine.*

Before replacing the turbine, carefully examine handpiece head for dents. If there are any dents in the head, return the entire handpiece to your dealer for repairs. **Do not install** replacement turbine into a dented handpiece. Installation will permanently damage turbine.

1. Remove cap by turning counterclockwise, using the cap wrench.
2. Remove worn turbine by pressing the blunt end of an inserted shipping post or bur against a hard surface to expel turbine.
3. Remove all foreign material and debris from interior of handpiece head. **Clean** and **lubricate** interior of head with Champ-Lube™ 20+.
4. Insert new turbine cartridge into handpiece head. When replacing with canister style cartridge, the index ball on the canister must be lined up with concave groove inside handpiece head to ensure proper fit.

*continued*

5. Replace cap using fingers to prevent cross threading. Once seated, tighten cap with cap wrench.
6. Install bur securely into handpiece and operate per handpiece instructions. ***Do not operate without a bur!***

### **Sterilization Procedures**

Remove the bur and any attachments from the handpiece.

Put in steam or chemical vapor sterilizer and cycle.

Autoclave/chemiclave up to 275° (135°C).

### **Lubricate Before Use**

After sterilization and cool down time, insert bur and tighten with bur tool. Lubricate with 1 or 2 drops of Champ-Lube 20+ and briefly operate to expel excess lubricant.

## ACCESSORIES

Champ-Lube 2/Pack _____	30010
Champ-Lube 4/Pack _____	30015
Champ-Lube Plus Oiler Pen _____	30001
Replacement Chuck _____	67000
Bur Wrench-Autoclavable _____	68010
Replacement Cap _____	69000
O-ring _____	100-410

# ATHENA CHAMPION™

2621 Corrinado Court • Fort Wayne, IN 46808  
(800) 253-1771 • (260) 373-1917 • FAX (260) 373-0395